

How to use the Comfortpan, bed pan.

Think of the pan as a bicycle seat. The "pointed" end goes toward the patient's feet. Place the seat part low under the hip area just so the pan will catch the waste. If the pan is placed too high up the back then the patient will be sitting in the pan or the edge of the seat will not be comfortable across the buttocks, so place it on the low side.

For best use, place a towel or hospital blue under-pad under the patient when you use the bedpan in case there are any spills. That will protect your bed linen.

When the patient is finished using the Comfortpan, grip the edge of it and keep it level as the patient rolls off of it and you carefully "scissor" it out from under them or else it will tip and spill.

Keep it level! If the patient is able to raise their hips then it is easily removed out toward their feet and away.....still keeping it level.

Carrying it to the toilet to empty is easy if you tip the liquid back under the seat. It empties and rinses easily and can be sprayed with a disinfectant spray or simply rinsed with warm soapy water.

If you find the Comfortpan difficult to remove from the patient's hip area, it can be dusted with some talc for easier removal, but the seat is textured for a non-stick removal.

If you have any questions at all, please give me a call (800) 317-8161 OR Email: info@comfortpan.com.

Warm regards,

Judi

Copyright © 2004-2013. CHURCH PRODUCTS, INC. All rights reserved